

Mobile Activity Tray

User Manual


Model #6500010X (Short)

Model #6500030X (Tall)

UMMAT06-22LIT




Parts

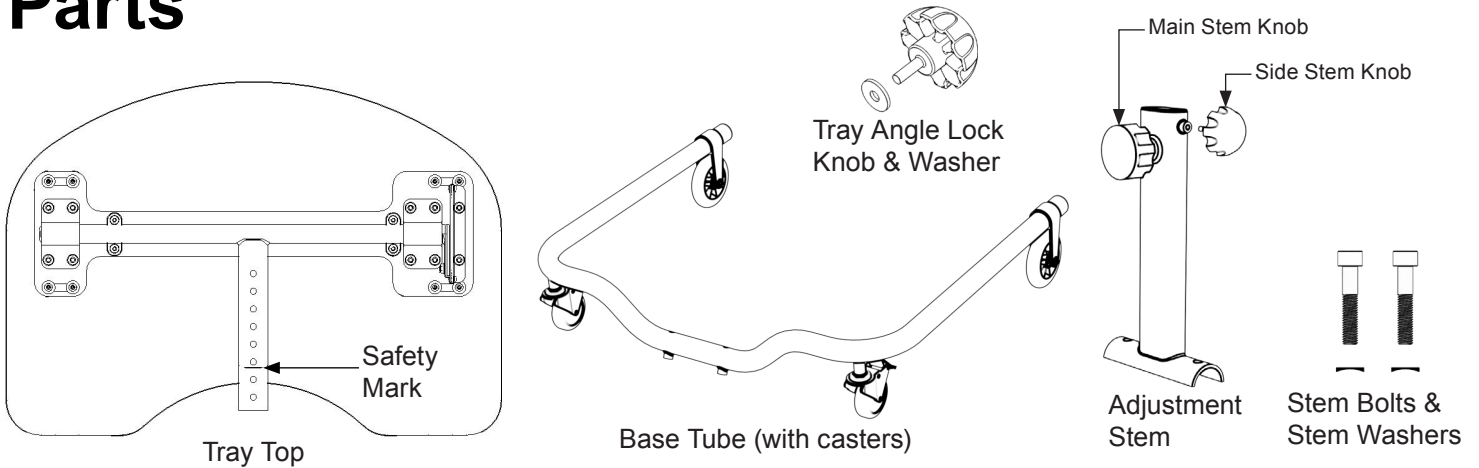


1. User Assembly

- A.** Attach the Adjustment Stem to the Base Tube. Using the 8mm Allen Wrench provided, install the Stem Bolts & Stem Washers as shown below. Ensure curved face of each Stem Washer is installed against curved surface of Adjustment Stem.
- B.** To install Tray Top, pull and hold Main Stem Knob outward while inserting Tray Top into Adjustment Stem (insert past Safety Mark). Release Main Stem Knob to lock Tray Top in place at desired height. Finally, tighten Main Stem Knob & Side Stem Knob to secure Tray Top.

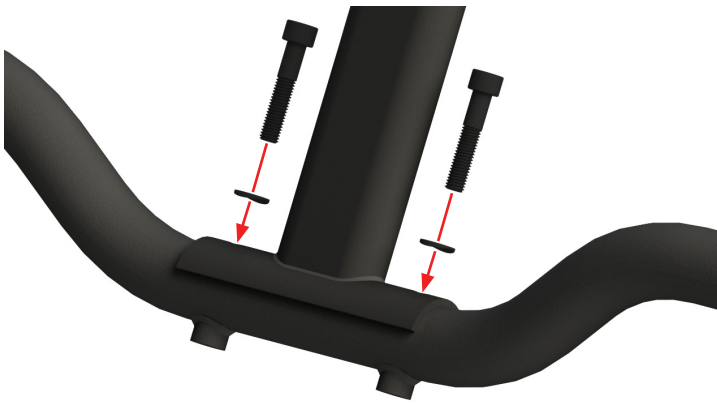

- C.** Install Tray Angle Lock Knob & Washer.

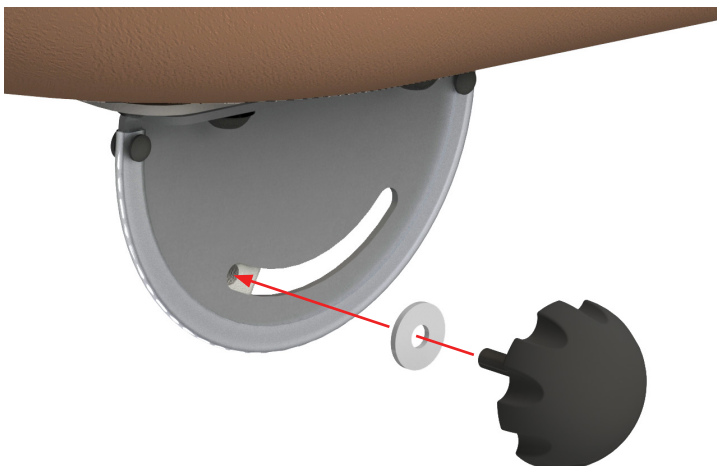


Intended Use


Designed with adjustability, maneuverability, and versatility in mind; the Mobile Activity Tray (MAT) is intended for a number of activities for children with disabilities, such as eating, playing, and classwork. The adjustable frame has lockable casters for stationary activities, and the tray itself has a wide tilt range and is coated in a durable elastomer skin that is grippy and easy to clean.

2. Adjusting Tray Height

A. Loosen Main Stem Knob & Side Stem Knob. Pull and hold Main Stem Knob outward to adjust tray height. Release Main Stem Knob to lock Tray Top in place at desired height. Finally, tighten Main Stem Knob & Side Stem Knob to secure Tray Top.

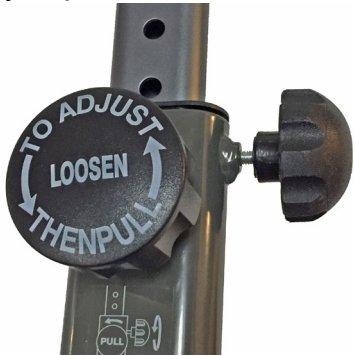


B. When adjusting Tray Top, never exceed Safety Mark.

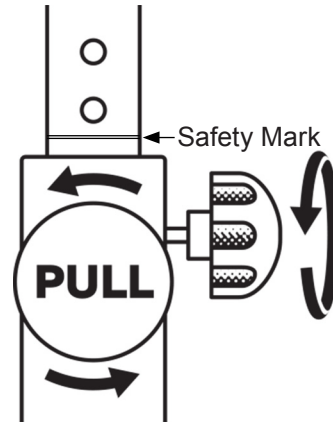

3. Adjusting Tray Angle

A. Loosen the Tray Angle Lock Knob, but do not remove. Tilt Tray Top to desired angle and re-tighten the Tray Angle Lock Knob.

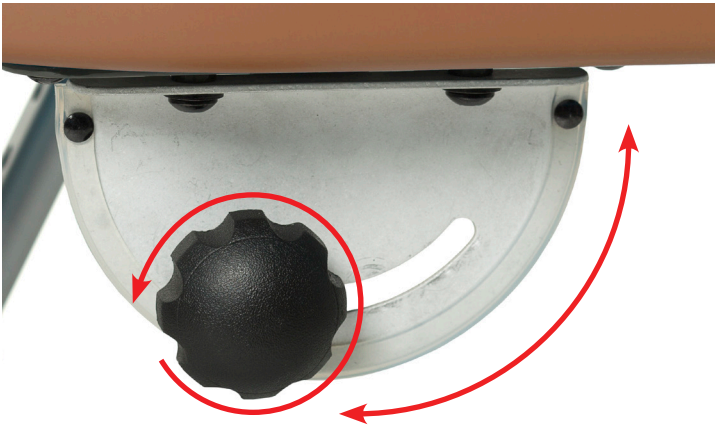

4. Using Caster Locks

A. Level position is unlocked.



B. Tilted position is locked.



Special Tomato® Customer Service

US & International Phone: (866) 529-8407

Fax: 1-315-429-8862

Email: info@bergeroncompanies.com

Website: www.specialtomato.com

2 YEAR LIMITED WARRANTY:

Special Tomato warrants to the original retail purchaser of the Mobile Activity Tray product, that if any part or component proves defective in material or workmanship within two years of the purchase date, the defective part will be repaired or replaced (at Special Tomato's discretion) free of charge. Warranty service may be performed by an authorized service center or at Special Tomato's discretion.

This warranty does not cover normal wear and tear, or damage caused by accident or misuse.

To exercise this limited warranty, the user should first obtain a Return Authorization Number from Special Tomato.

The product must be delivered with charges pre-paid (UPS recommended) to the factory or to an authorized service center, together with a copy of the original invoice, the Return Authorization Number and a written description of the problem.

NOTICE:

Use only Special Tomato accessories and parts on Special Tomato products. Special Tomato parts are not interchangeable with other manufacturers' products. Immediately replace any worn parts for safety.

Maintenance and Environments for Use

Clean with mild detergent/cleaner but do not submerge.

Moving parts may be lubricated with a high quality PTFE or similar dry-lubricant spray. Do not use WD-40 or silicone lubricants.

MAT intended for classroom and at-home use. Light use in outdoor settings only.

⚠️ WARNING:

- MAT is not for use as a walking aid. It is not intended to support full weight of user.
- Never leave child unattended
- Due to tipping, do not overload any part of the MAT surface.
- Keep product away from extreme sources of heat
- Keep away from mud, rain, snow, etc. and clean when soiled. Dry when wet.



 BERGERON HEALTH CARE

Bergeron Health Care
15 South Second Street
Dolgeville, New York 13329
1-866-529-8407



Global Carehab
HOLMSTRUPGAARDVEJ 11
8220 BRABRAND
DENMARK
+45 24 65 55 50



© 2022 Bergeron By Design
The information contained herein
is subject to change without notice.